


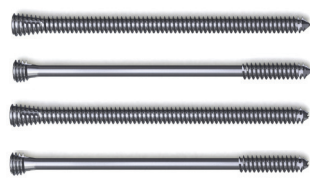
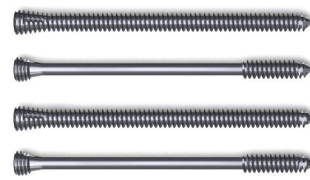
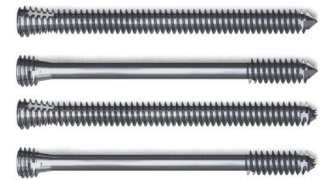
Exclusively foot & ankle
Paragon²⁰[®]



Joust™ Beaming Screw System
PRODUCT INFORMATION



Joust™ Beaming Screw System

	5.0 mm Beam	5.5 mm Beam	7.2 mm Beam
			
Lengths	50 —120 mm (5 mm increments)		65 —185 mm (5mm increments)
Solid and Cannulated Offerings	Yes	Yes	Yes
Fully and Partially Threaded Offerings	Yes	Yes	Yes
Total Beams	60	60	100


Cannulated and Solid Beams: 5.0 mm, 5.5 mm and 7.2 mm

- Constructed from Type II Anodized Ti-6Al-4V ELI for improved fatigue strength ¹
- Headless to minimize prominence and avoid impingement
- Sharp tip to help aid in ease of insertion
- Offered partially threaded to allow for compression or fully threaded for increased thread purchase
- Beam lengths optimized to accommodate varying patient anatomy and to allow surgeons greater flexibility in final placement
- Beams available in three diameters to allow surgeons to beam the first and lesser rays

Joust™ Beaming Screw System

SYSTEM SPECIFIC INSTRUMENTATION

Joust™ PRECISION® Guide (patent pending)

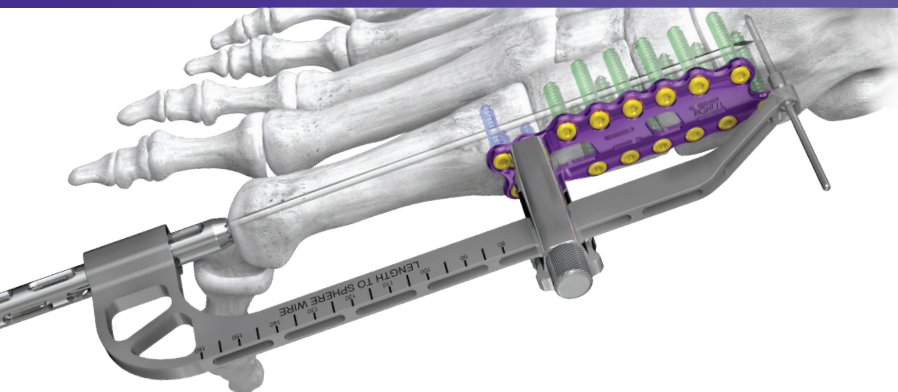

- The Joust™ PRECISION® Guide is provided to aid surgeons in placing a medial column beam at a desired trajectory and endpoint
- The patent pending PRECISION® Guide helps position the K-wire from the head of the first metatarsal to the talus and allows the plate to set the trajectory for a beam to pass through without hitting any on-axis plate screws in the construct
- For use with the 7.2 mm beam only


Joust™ Beaming Screw System

JOUST™ BEAM AND GORILLA® STRADDLE PLATE




- Joust™ Beams may be used in conjunction with a 2.0 mm thick Gorilla® Straddle Plate to further reinforce the construct
- The Straddle Plate's thickness and height are optimized to resist bending, while assisting in alignment of the medial column
- 10 Gorilla® Straddle Plates are available in left and right configurations in sizes ranging from extra small to extra large

- The PRECISION® Guide may be used in conjunction with the Straddle Plate and medial column beam to avoid interference between the plate screws and beam

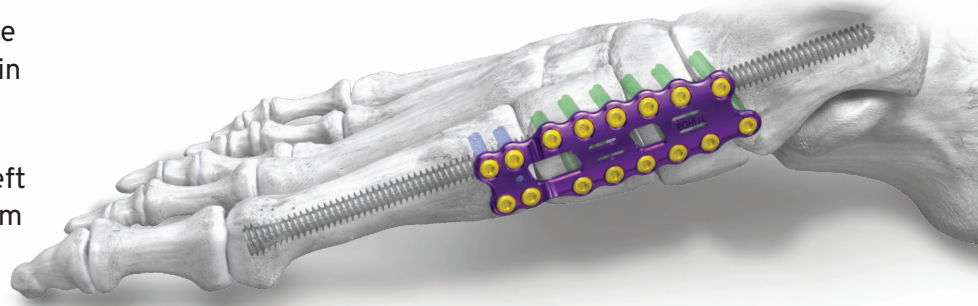




Beaming Instrumentation

- Each beam has a dedicated countersink to allow the surgeon to insert the beam beneath the surface of the bone
- A 5.0 and 5.5 mm tap are included to allow surgeons to determine the appropriate diameter of beam to be used in the lesser rays
- A standard and long drill are included for use with the 7.2 mm beams to accommodate varying patient anatomy and surgeon preference

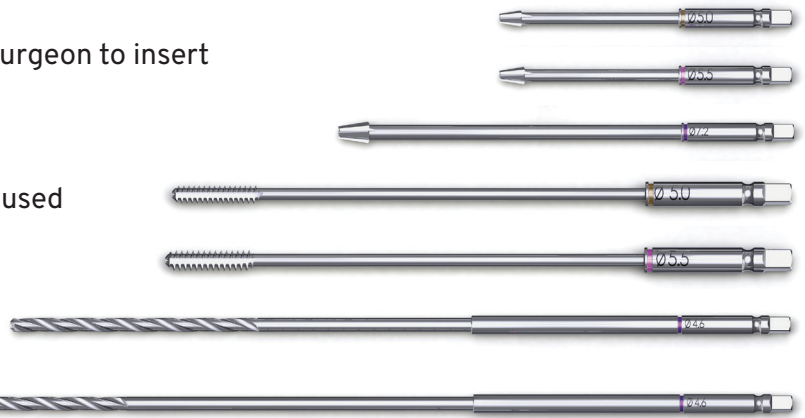

ALTERNATIVE FIXATION OPTIONS AND ASSOCIATED BIOLOGICS:

- Gorilla® Lateral Column Plating System
- ParaDerm™ Dermal Matrix
- Gorilla® Medial Column Plating System

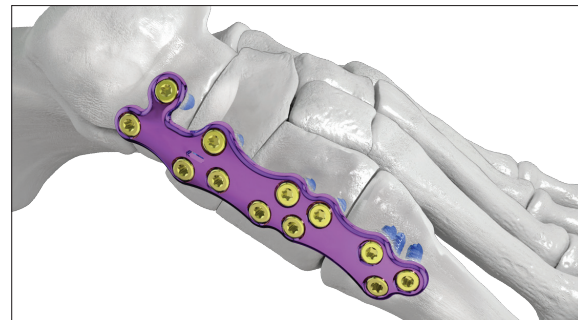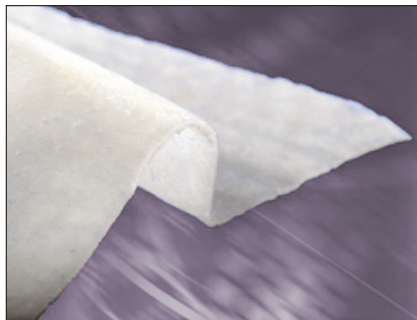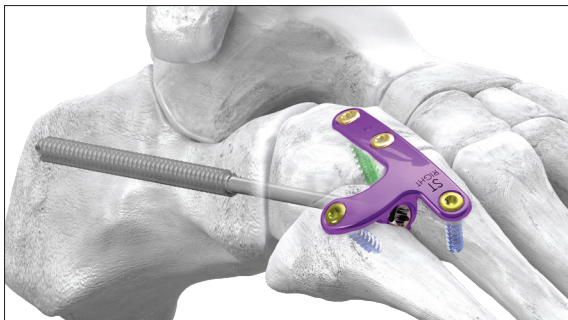


JBSS-01 RevA

™ Trademarks and ® Registered Trademarks of Paragon 28, Inc.
© Copyright 2018 Paragon 28, Inc. All rights reserved.
Patents: www.paragon28.com/patents

References:

¹ Whitten, Andy. Evaluation of the Effects of Anodization on the Fatigue Performance of Titanium Alloy. *Fatigue and Fracture of Medical Metallic Materials and Devices*, STP 1559. West Conshohocken, PA: ASTM International; 2013: 109-121.

Paragon 28, Inc. 
4B Inverness Ct. E., Suite 280
Englewood, CO 80112 USA
(855) 786-2828

Paragon 28 Medical Devices Trading Limited
43 Fitzwilliam Square West
Dublin 2, D02 K792, Ireland
+353(0) 15414756



For additional information on Paragon 28® and its products please visit www.paragon28.com

